CareMat

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CareMat

with 868 MHz EasyWave® transmitter

The pressure-sensitive bed mat for dementia patients and persons at risk of falling.

Translation of the original instructions

1 Safety instructions

These instructions must be completely read through prior to the commissioning and use of the CareMat. They should be kept safely for future use. Warranty claims will be invalid in the event of any damage to the product, to goods or persons that is due to failure to observe these instructions.

Instructions for nursing staff

- The CareMat operates purely as a support for the nursing staff, but is not a substitute for them.
- The functional correctness of the CareMat is to be checked each time that it is used.

Notes for the installer

- The electrical installation work and commissioning of the product may only be carried out by a qualified technician. Follow ESD protection rules.

2 Product description

2.1 Described radio system


The CareMat with 868 MHz EasyWave® radio transmitter is, compatible with the ELDAT RCL07 radio receiver. The RCL07 can receive radio signals from different ELDAT radio transmitters and relay them to a cable connected call system. Detailed information on this and further compatible 868 MHz EasyWave® appliances is available on the following website: www.eldat.de

2.2 Areas of application and function


The CareMat was primarily designed for use in retirement and nursing homes, hospitals and psychiatric wards as well as for sheltered residential accommodation. The necessary care and supervision of patients with dementia and of those who are at risk of falling represents a significant increase in care expenditure for care practitioners. The CareMat is the ideal support: Risks of falling and the dangers which arise from wandering or loss of orientation can be significantly reduced.

The CareMat as a pressure-sensitive sensing device is laid directly in front of the care bed. The RCL07 is connected to the call system via the plug-in connection of the call button. As soon as the person leaves the bed and steps onto the CareMat, this switches on and sends out an alarm signal to the call system via the RCL07. The wireless signalling system reduces the hazards due to a tangle of wiring around the care bed.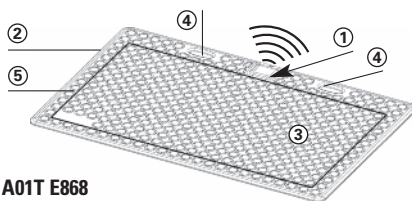


CareMat A01T E868
CareMat A11T E868
1100 x 700 mm

- ① Housing with radio transmitter
- ② Chamfered, wheelchair-accessible edges
- ③ Non-slip surface with stud texture design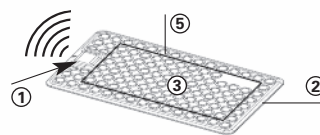


CareMat B01T E868
CareMat B11T E868
700 x 400 mm

- ④ Carrying handle
- ⑤ Active switching zone
- ⑥ RCL07-radio receiver (optional component)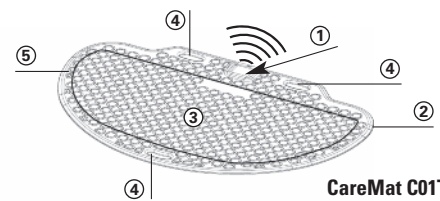


CareMat C01T E868
CareMat C11T E868
1100 x 700 mm,
semi-circular

3 Day to day operation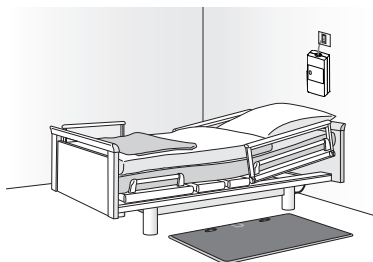

Align the CareMat centrally in front of the bed with the handles underneath the edge of the bed so that missing the mat is impossible.

RCL07 radio receiver connected to the call system.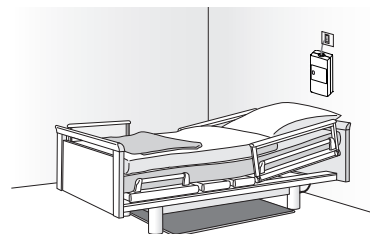

Should the CareMat not be needed, it can be pushed underneath the bed.

4 Cleaning and disinfecting

- The CareMat can be cleaned using commercially available disinfecting and cleaning agents with max. 70 vol% alcohol. No corrosive agents should be used.
- The CareMat is neither autoclavable nor can it be disinfected using steam.


5 Transport and storage

- During transport and storage, any significant bending of the CareMat should be avoided.
- Store the CareMat in a dry, clean and flat location.
- Do not place any heavy loads on the CareMat and do not stack more than 10 packed mats on top of each other.

6 Radio system combined with cable-connected light signalling system

6.1 Configuration RCL07 radio receiver

The plug connection of the RCL07 to the call system must be installed by a qualified technician. One CareMat and one RCL07 radio receiver are installed per room. If anyone steps on an assigned CareMat, an alarm signal is transmitted to the RCL07. The LED illuminates and the relay output reacts. The CareMat continues to transmit for as long as it is activated and in any case for a maximum of 10 seconds. The LED does not illuminate in the stand-by mode. The deadman's operation is factory set.

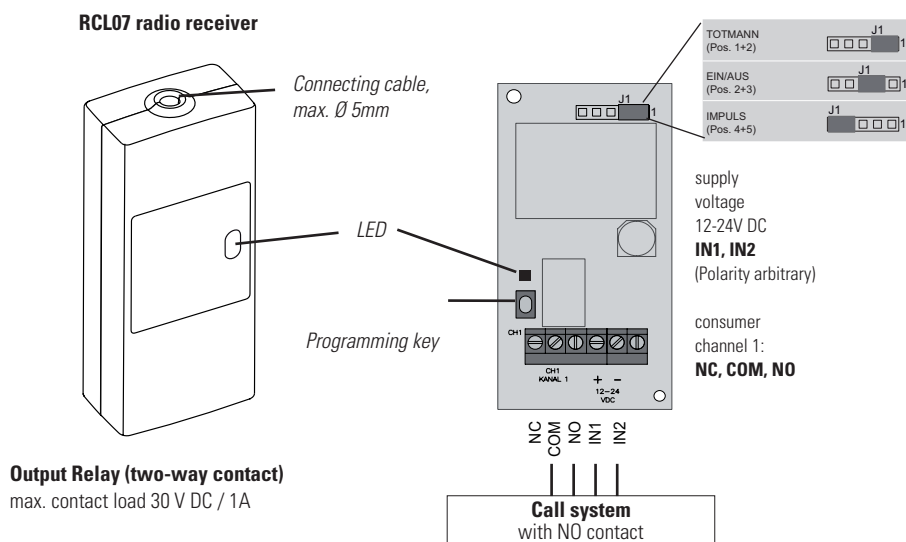6.2 Programming the radio transmitter

To tune-in the CareMat on the RCL07: Press the programming key briefly (< 1.6 seconds). The LED then flashes. Tune-in the CareMat by actuating it. When the programming is successfully completed the LED illuminates. To return to the operating mode briefly press the programming key once more.

To switch-off CareMat on RCL07: press the programming key for longer than 1.6 seconds until the LED starts to flash rapidly. The switching-off mode is activated. Then actuate CareMat. The LED illuminates briefly. The CareMat is now switched-off. To return again to the operating mode, press the programming key briefly.

⚠ Note: The CareMat and the signal transmitter form a call system. They should be clearly identified so that they cannot be mixed up. In this way, it can be ensured that the call is only triggered in the particular room where the CareMat and the RCL07 are also actually located.

7 Maintenance and fault finding

Apart from battery changing the CareMat is maintenance-free. Only tighten the screws lightly. If despite this, function is not being provided, the following checks should be made:

- Is the call system operating and the RCL07 correctly connected?
- Is the CareMat correctly tuned-in to the RCL07?
- Is the signal transmitter battery OK?

If the points mentioned above have been checked out, but the functions of the CareMat are, despite this, still not being provided, you should consult a qualified expert.

9 Technical specifications

Article	CareMat Ax1T E868 / Bx1T E868 / Cx1T E868	Signaling protocol	ELDAT EasyWave®
Dimensions (LxWxH)	1100x700x9 mm / 700x400x9 mm / 1100x700x9 mm, height with housing 15 mm	Radio frequency	868.30 MHz
Weight [kg]	8.2 kg / 3.2 kg / 6.5 kg	Working life	Dependent on application approx. 2 years
Material	Polyurethane (PU)	Radio range	up to 30 m in buildings, dependent on structural building conditions (up to 100 m in free field)
Colour	Careyellow, Caregrey	Type of battery	1x CR2032, 3V Lithium
Surface	Stud texture design	Type of protection	IP54
Weight of persons	At least 10 kg	Ambient temperature	0 up to +55° C
Chemical resistance	Water (room temp.), body fluids, customary disinfectants, max. 70 vol% alcohol	<i>Subject to technical modifications</i>	

10 EC-Declaration of Conformity

Manufacturer: Bircher Reglomat AG, Wiesengasse 20, CH-8222 Beringen
 Following directives have been observed: RoHS 2011/65/EU, RED 2014/53/EU
 Product variants: CareMat Ax1T E868, CareMat Bx1T E868, CareMat Cx1T E868, CareMat Mx1T E868

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