CareMat

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CareMat

with 869.2125 MHz VarioMobil® radio transmitter or
with 869.2375 MHz AAL Profiline® radio transmitter
(social alarm frequencies in accordance with DIN EN 50134)

BIRCHER

Reglomat

ENGLISH

Pressure-sensitive bedside mat
to detect patients who wander or fall

Translation of the original instructions

1 Safety instructions

These instructions must be read thoroughly prior to installation and use of the CareMat, and should be retained for future reference. Warranty claims will be invalid in the event of any damage to the product, to goods or persons that is due to failure to observe these instructions.

Instructions for nursing staff

- CareMat operates purely as a support for nursing staff, not as a substitute for them.
- Functionality of the CareMat should always be checked prior to use.

Notes for the installer

- The electrical installation work and commissioning of the product may only be carried out by a qualified technician. Follow ESD protection rules.

2 Product description

2.1 Radio system description


The CareMats with 869.2125 MHz VarioMobil® radio transmitters are compatible with the VarioRec® radio receivers from Lehmann Electronic GmbH. The CareMats with 869.2375 MHz AAL Profiline® radio transmitters are compatible with the AAL Profiline® radio receivers from Lehmann Electronic GmbH. The radio receivers (not part of the scope of delivery) can receive the radio signals from the radio transmitters on the social alarm frequency bands approved for this purpose, and relay them to a cable-connected or wireless call system, for example. Detailed information about this device and other compatible social alarm frequency devices can be found on the following websites:

www.lehmannweb.de and www.aal-homecare.com

2.2 Areas of application and function

The CareMat is designed for use in nursing homes, hospitals, psychiatric wards, and home care. Care and supervision of accident-prone patients and those with dementia require significant time and financial investments from caregivers. CareMat is an ideal addition to reduce risks that may arise from wandering patients.

The CareMat with radio transmitter is placed beside a patient's bed, as a pressure-sensitive detector. The radio receiver is connected to the existing call system. As soon as a person gets out of the bed and steps on to the CareMat, the mat sends a signal so the call system can trigger an alarm. The wireless radio system facilitates handling around the patient's bed.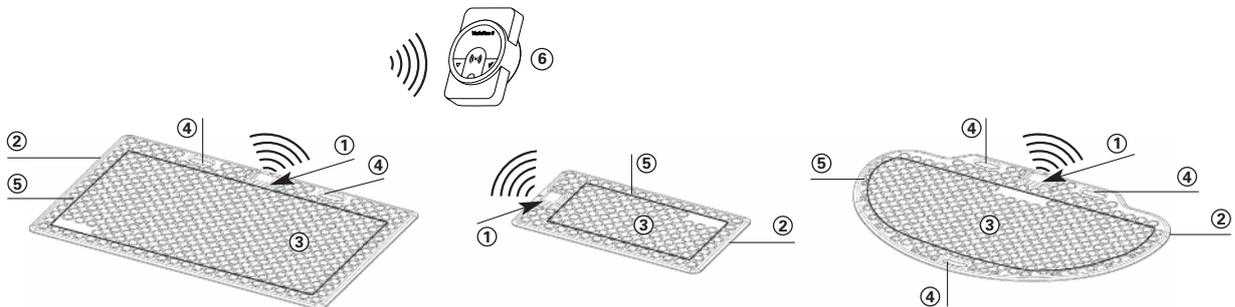

CareMat A01T L869 and A01T L869.23
CareMat A11T L869 and A11T L869.23
1100 x 700 mm


CareMat B01T L869 and B01T L869.23
CareMat B11T L869 and B11T L869.23
700 x 400 mm

CareMat C01T L869 and C01T L869.23
CareMat C11T L869 and C11T L869.23
1100 x 700 mm,
semi-circular

- ① Housing with radio transmitter
- ② Chamfered, wheelchair-accessible edges
- ③ Non-slip surface with stud texture design


- ④ Carrying handle
- ⑤ Active switching zone
- ⑥ VarioRec® or AAL Profiline® radio receivers (not part of the scope of delivery)

3 Day to day operation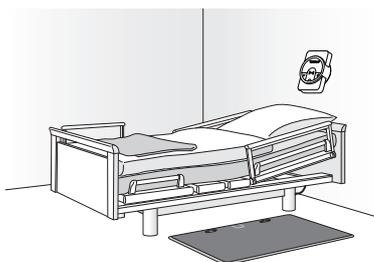

CareMat should be centrally aligned next to the bed so it is impossible to bypass.

Connect the associated radio receiver to the call system.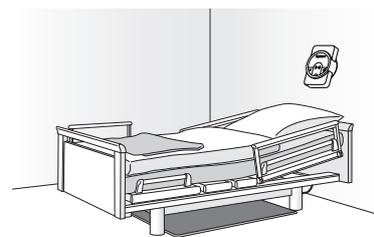

If the CareMat is not needed during the day, it can be stored underneath the bed.

4 Cleaning and disinfecting

- The CareMat can be cleaned using commercially available disinfecting and cleaning agents with max. 70 vol% alcohol. No corrosive agents should be used.
- The CareMat is neither autoclavable nor can it be disinfected using steam.

5 Transport and storage

- Do not bend CareMat during transport, storage or installation.
- CareMat must be stored flat in a clean, dry location with temperature range 0 °C to +55 °C.
- Do not place any heavy loads on the CareMat and do not stack more than 10 packed mats on top of each other.

6 Radio system combined with cable-connected light signalling system

6.1 CareMat configuration


A CareMat with integrated VarioMobil® or AAL Profiline® radio transmitter can be combined with different radio receivers with the same frequency as described in 2.1. Radio receivers do not form part of the scope of delivery. The performance features offered by each radio receiver and the configuration can be found in the technical descriptions provided by the manufacturer, Lehmann Electronic GmbH. These can be found online at www.lehmannweb.de or www.aal-homecare.com.

A radio receiver must be installed in the call system by a qualified technician.

! Note: Once configured, a CareMat with radio transmitter and a radio receiver form a call system. Particularly when several of these systems are in use in adjacent rooms, the individual systems should be identified clearly to ensure that components cannot be mixed up between rooms.

6.2 Replacing radio transmitter battery

Remove housing cover on rear
(6 countersunk screws M3x5, cross-head size PH1)

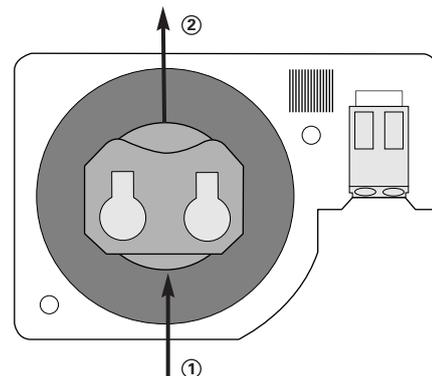- ① With a thin, pointed object (preferably plastic), push the battery out of the housing compartment.
- ② Insert the new battery, ensuring the positive (+) side is facing you.

Reinstall housing cover, carefully tighten the 6 screws by hand, do not overtighten.

7 Maintenance and troubleshooting

The CareMat is maintenance-free until the batteries need to be replaced. Ensure battery compartment screws are not over-tightened.

If batteries have been replaced and the CareMat is still not functioning properly, check the following:

- Is the call system operating and the radio receiver correctly connected?
- Is the CareMat correctly taught in to the radio receiver?
- Is the signal transmitter battery OK?

If the CareMat is still not functioning after the above have been checked, consult a qualified expert.

8 Disposal

The signal transmitter and its battery are to be disposed of at a suitable collecting point, so that the components are recycled, correctly treated and disposed of in an environmentally correct manner.

The CareMat minus the signal transmitter and battery can be disposed of as household refuse. You can obtain further information on your local collection point from your refuse collection authority or from the competent representative.

9 Technical specifications

Article	CareMat Ax1T L869 / Bx1T L869 / Cx1T L869 CareMat Ax1T L869.23 / Bx1T L869.23 / Cx1T L869.23	Radio protocols	Lehmann Electronic GmbH, VarioRec® Lehmann Electronic GmbH, AAL-Profiline®
Dimensions (LxWxH)	1100x700x9 mm / 700x400x9 mm / 1100x700x9 mm, height with housing 15 mm	Radio frequency (Social alarm frequencies)	CareMat xx1T L869: 869.2125 MHz CareMat xx1T L869.23: 869.2375 MHz
Weight [kg]	8.2 kg / 3.2 kg / 6.5 kg	Working life	Dependent on application approx. 2 – 3 years
Material	Polyurethane (PU)	Radio range	up to 30 m in buildings, dependent on structural building conditions (up to 100 m in free field)
Colour	Careyellow, Caregrey	Type of battery	1x CR2032, 3V Lithium
Surface	Stud texture design	Type of protection	IP54
Weight of persons	At least 10 kg	Ambient temperature	0 to +55° C
Chemical resistance	Water (room temp.), body fluids, customary disinfectants, max. 70 vol% alcohol	<i>Subject to technical modifications</i>	

10 EC-Declaration of Conformity

Manufacturer:
Following directives have been observed:
Product variants:

Bircher Reglomat AG, Wiesengasse 20, CH-8222 Beringen
RoHS 2011/65/EU, RED 2014/53/EU
CareMat Ax1T L869, CareMat Bx1T L869, CareMat Cx1T L869, CareMat Mx1T L869
CareMat Ax1T L869.23, CareMat Bx1T L869.23, CareMat Cx1T L869.23, CareMat Mx1T L869.23

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