CareMat

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CareMat

with 2.4 GHz Ekahau A4 WiFi Tag

The pressure-sensitive bedside mat for persons with dementia and who are prone to falls

Translation of the original operating instructions

1 Safety instructions

Read these operating instructions carefully and keep them for future reference before putting CareMat into operation. CareMat can be used in addition to call devices and systems but does not replace them.

Observing the operating instructions is a prerequisite for safe operation of CareMat and for enjoying the indicated product properties and performance features. Bircher Reglomat AG does not accept any liability for personal injury, damage to property or financial loss resulting from non-observance of the operating instructions. Liability for material defects is excluded in such cases.

Instructions for specialist nursing staff

- CareMat provides support to but does not replace specialized nursing staff.
- CareMat must be tested prior to use by trained personnel.


Instructions for the installer

- The electrical installation and start-up of the product may only be carried out by a trained call system installer.


2 Product description

2.1 Description of the radio system


CareMat with an integrated 2.4 GHz Ekahau A4 WiFi Tag is designed for wireless call systems. CareMat teach-in is accomplished using the relevant MAC address and can be integrated into the institution's existing radio network.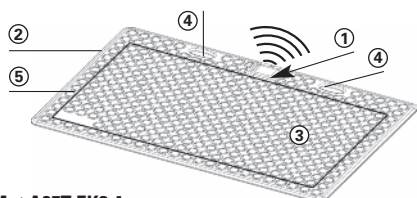

CareMat A05T EK2.4
CareMat A15T EK2.4
1100 x 700 mm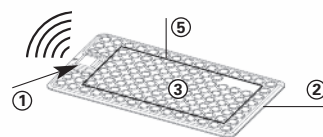

CareMat B05T EK2.4
CareMat B15T EK2.4
700 x 400 mm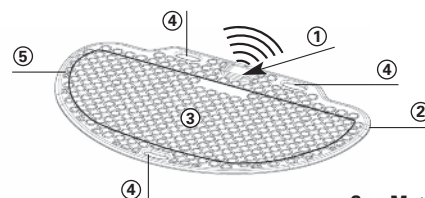

CareMat C05T EK2.4
CareMat C15T EK2.4
1100 x 700 mm,
semi-circular

- ① 2.4 GHz Ekahau A4 WiFi Tag
- ② Chamfered, wheelchair-accessible edges
- ③ Non-slip surface with stud texture design


2.2 Application areas and functions

CareMat was primarily designed for use in retirement and nursing homes, in hospitals and psychiatric wards as well as in assisted living facilities. The necessary care for and supervision of persons who have dementia or who are prone to falls increases the workload of the nursing staff. CareMat provides ideal support: The risk of falls and the dangers which can arise from patients wandering off or disorientation can be considerably reduced.

CareMat is placed directly beside the patient's bed, as a pressure-sensitive detector. As soon as the person leaves the bed and steps on the CareMat, it is triggered and sends an alarm to the call system via the WiFi tag. The wireless radio system facilitates handling around the patient's bed.


- ④ Carrying handle
- ⑤ Active switching zone
- ⑥ Access point

3 Daily operation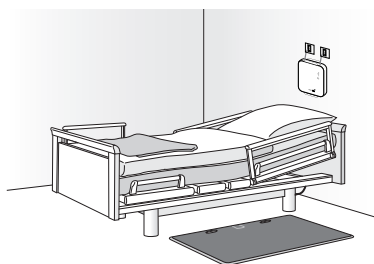

Centre CareMat in front of the bed with the handles underneath the edge of the bed, leaving enough space to walk around the mat.

Connect the radio receiver to the call system.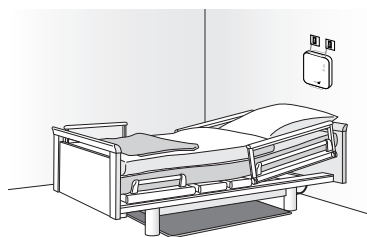

CareMat can be pushed under the bed whenever it is not needed.






4 Cleaning and disinfection

- Clean CareMat only with commercially available disinfectants and cleaning agents with a maximum alcohol content of 70% vol. Do not use corrosive agents.
- Do not use an autoclave or steam to disinfect CareMat.
- Mechanical cleaning agents are not permitted.

5 Transport and storage

- Avoid excessive bending during transport and storage.
- Store CareMat in its original packaging, in a dry and clean place and in an upright position.
- The carrier handles are designed for better handling of CareMat in front of the patient's bed. These carrier handles are not intended for hanging CareMat and may cause irreversible deformation of CareMat over time.

6 Battery replacement

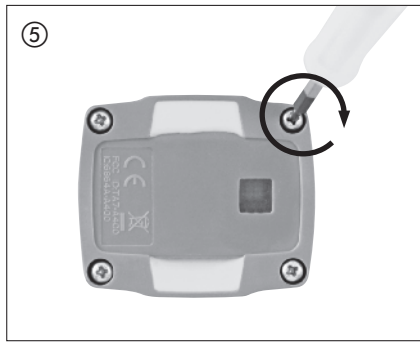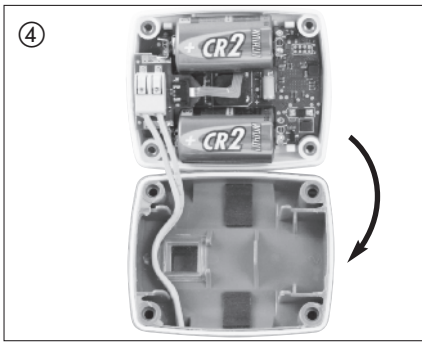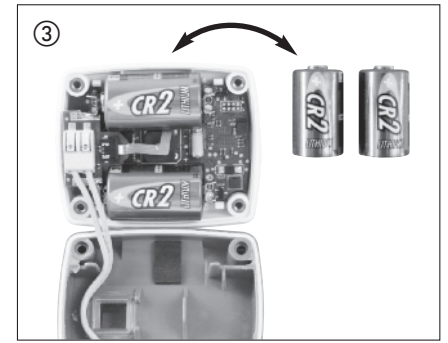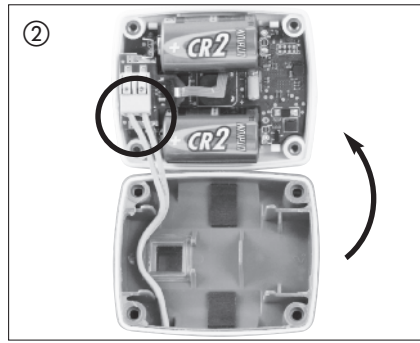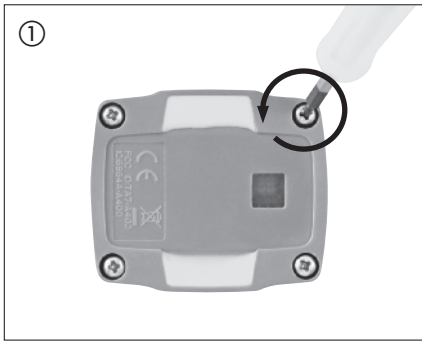1. *Underside of the mat*
Remove the four housing screws (Philips #1)
Caution: do not lose the four O-ring seals.
2. *Upper side of the mat*
Carefully raise the housing cover
Caution: Do not pull the wires out of the clamp terminals.
3. Replace the two CR2 batteries.
Caution: Observe correct polarity.
4. Place the housing cover back into position.
Caution: Take care to ensure that no foreign particles or moisture find their way into the housing or onto the sealing edge surrounding the housing and that no cables are trapped.
5. *Underside of the mat*
Restore the four screws together with the O-ring seals to their previous positions and carefully tighten them by hand.

7 Maintenance and trouble shooting

Maintenance, repair and trouble shooting may only be performed by a trained call system specialist. Aside from normal battery replacement, CareMat is maintenance-free.

8 Disposal

CareMat must not be disposed of as household waste. Recycle CareMat as appropriate.

9 Technical data

Article	CareMat Ax5T	CareMat Cx5T	CareMat Bx5T	CareMat Mx5T	Radio transmitter	Ekahau A4 WiFi Tag
Dimensions (LxWxH)	1100 x 700 x 19 mm	1100 x 700 x 19 mm	700 x 400 x 19 mm	283 x 202 x 19 mm	Radio frequency	802.11 b/g 2.4 GHz WiFi networks
Weight	8.2 kg	6.5 kg	3.2 kg	0.6 kg	Wireless range	up to 40 m @ 11 Mbit/s in buildings, dependent on structural building conditions (up to 60 m @ 2 Mbit/s in free field)
Material	Polyurethane (PUR)				Battery type	2 x 3 V lithium round cell CR2
Colour	Mat: Careyellow, Caregrey				Battery life	18 months (in 24 h with 4 test intervals / 2 position changes / 3 calls triggered)
Surface	Active grip structure				Protection class	IP54
Weight of person	at least 10 kg				Ambient temperature	0 to +50°C
Chemical resistance	Water (room temperature), bodily fluids, commercially available disinfectants, max. 70% vol. alcohol					
<i>Technical specifications subject to change</i>						

10 EC-Declaration of Conformity

Manufacturer:
Following directives have been observed:
Product variants:

Bircher Reglomat AG, Wiesengasse 20, CH-8222 Beringen
RoHS 2011/65/EU, RED 2014/53/EU
CareMat Ax5T EK2.4, CareMat Bx5T EK2.4, CareMat Cx5T EK2.4, CareMat Mx5T EK2.4

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